

QuantiFERON-TB Gold Plus Blood Collection Tubes Pre-Analytical Attestation

This Attestation is used to confirm that the pre-analytical stages of the QuantiFERON-TB Gold Plus Blood Collection Tubes were processed in accordance with the requirements of the Qiagen QuantiFERON-TB Gold Plus Blood Collection Tubes Package Insert.

Complete the form and submit this document to Laboratory Services with each patients' QuantiFERON-TB Gold Plus Blood Collection Tubes.

Directions:

Check the box below to confirm each step was completed within the specified ranges.

1.	<input type="checkbox"/>	QFT-Plus Blood Collection Tubes were transferred to a $37^{\circ}\text{C} \pm 1^{\circ}\text{C}$ incubator as soon as possible and within 16 hours of blood collection. <i>Important: QFT-Plus Blood Collection Tubes should be at room temperature $17\text{--}25^{\circ}\text{C}$ ($62.6\text{--}77^{\circ}\text{F}$) at the time of blood collection.</i>
2.	<input type="checkbox"/>	The QFT-Plus Blood Collection Tubes were incubated UPRIGHT at $37^{\circ}\text{C} \pm 1^{\circ}\text{C}$ for 16 to 24 hours. <i>Note: After incubation, blood collection tubes may be held between 4°C to 27°C for up to 3 days pre or post centrifugation.</i>
3.	<input type="checkbox"/>	After incubation of the tubes at $37^{\circ}\text{C} \pm 1^{\circ}\text{C}$, the plasma tubes were centrifuged for 5–15 minutes at 2000 to 3000 RCF (g). <i>Note: Plasma samples can be stored in centrifuged QFT-Plus Blood Collection Tubes for up to 28 days at $2\text{--}8^{\circ}\text{C}$ before testing.</i>

<https://www.qiagen.com/us/resources/resourcedetail?id=efe732d5-4b93-477f-983b-7390d0a14637&lang=en>

I have read the requirements for the pre-analytical stages of the QuantiFERON-TB Gold Plus Blood Collection Tubes. I attest that the specimens included in this shipment have been processed accurately according to Qiagen's requirements.

Facility Name/Code:	
Signature:	
Printed Name:	
Date:	